wellion[®] AMADEUS

**HANDBUCH/
OWNER'S GUIDE**

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Dear Wellion AMADEUS Blood Glucose Meter Owner,

Thank you for choosing the Wellion AMADEUS Blood Glucose Meter for monitoring your blood glucose levels with minimal effort.

Your new Wellion AMADEUS Blood Glucose Meter is an auto-coding system that does not require any manual coding thus significantly reducing testing errors. All of the information you need to use and maintain your new Wellion AMADEUS Blood Glucose Meter is provided in this manual.

The Wellion AMADEUS Blood Glucose Meter has many user friendly features to help you easily manage diabetes, such as hypoglycemia/hyperglycemia warnings, before/after meal marking, and asterisk marking. The test result from the AMADEUS Testing System is plasma-calibrated and thus allows for easy comparison to lab results

For questions and inquiries, please contact the authorized representative.

Table of Contents

INTENDED USE	55
IMPORTANT SAFETY INSTRUCTIONS	55
YOUR NEW MEASURING KIT	56
Wellion AMADEUS Blood Glucose Meter	57
Meter Display Screen	59
The Wellion AMADEUS Blood Glucose Test Strip	60
SETTING UP YOUR NEW SYSTEM	61
Inserting (or Changing) the Batteries	61
Setting the Clock	62
Setting the Alarm	65
Turning ON/OFF Tone Sound	67
Setting Warning Values	68
Enable/Disable QC (Quality Control) Lock Mode	71
CONTROL SOLUTION TESTING	72
Performing a Glucose Control Solution Test	72
Control Solution Trouble Shooting	77
TESTING YOUR BLOOD GLUCOSE	78
Preparing your Lancing device	79
Important Information on Alternative Site Testing (AST)	81
Performing a Blood Glucose Test	82
UNDERSTANDING YOUR BLOOD GLUCOSE TEST RESULTS	87

RETRIEVAL OF RESULTS FROM THE DATA MEMORY	88
Upload Data	90
MAINTENANCE	90
Cleaning the Meter	90
Storage and Precautions	91
TROUBLESHOOTING	92
SYMBOLS	95
SPECIFICATIONS	96
MANUFACTURER'S WARRANTY	97

INTENDED USE

The Wellion AMADEUS Blood Glucose Meter is intended for quantitative measurement of glucose in venous whole blood, neonatal whole blood, or capillary whole blood drawn from fingertips, palm, or forearm.

It is intended for *in vitro* diagnostic use by people with diabetes or by health care professionals, as an aid to monitor glucose levels in diabetes mellitus. It is not to be used for diagnosis or screening of diabetes.

IMPORTANT:

- Dehydration – Severe dehydration may lead to inaccurate blood glucose test result. If you suspect you are severely dehydrated, contact your healthcare professional immediately.
- Hematocrit range - A hematocrit range that is higher than 70% or lower than 10% can cause inaccurate blood glucose test results.

IMPORTANT SAFETY INSTRUCTIONS

- The meter and lancing device should be cleaned and disinfected appropriately before use especially if the meter is being operated by a second person who is providing testing assistance to patients.
- All parts of the kit are considered biohazardous and can potentially transmit infectious diseases, even after cleaning and disinfection.

- Users should wash hands thoroughly with soap and water after handling the meter, the lancing device or test strips.


YOUR NEW MEASURING KIT

The Wellion AMADEUS Blood Glucose Meter kit includes the following items:

Wellion AMADEUS Blood Glucose Meter incl. batteries

Pouch

Owner's guide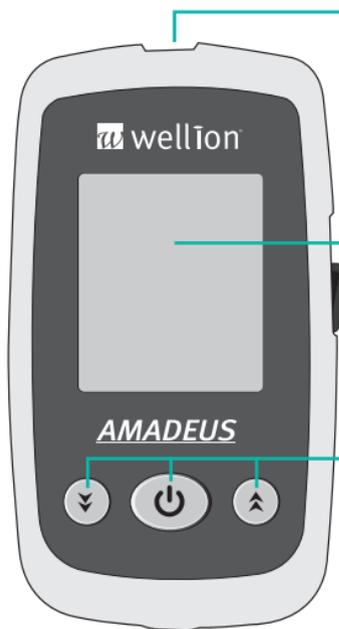

Test Strip Port

Insertion of the test strip - the meter turns on automatically.


Meter Display

Display of your blood glucose test results, memory values and other messages


Enter and buttons

Press and hold Enter () for 2 seconds to turn the meter on or off, or press Enter () to go into setup mode when not testing.

Press  or  to enter control mode with test strip inserted, during meter setup, or to navigate through stored test values.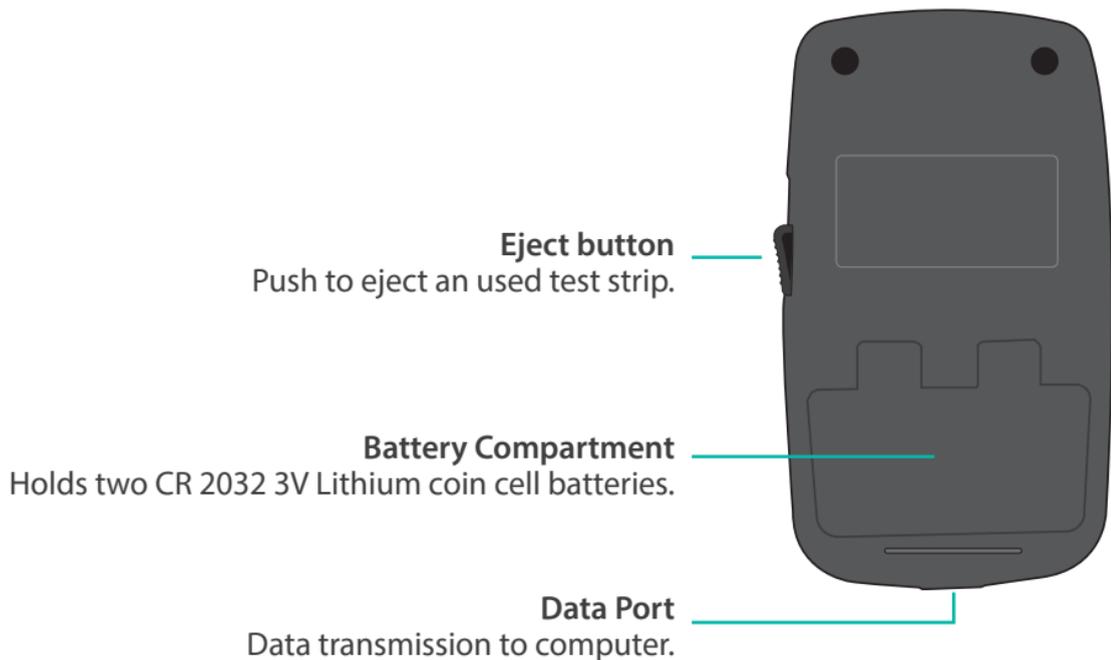

Meter Display Screen

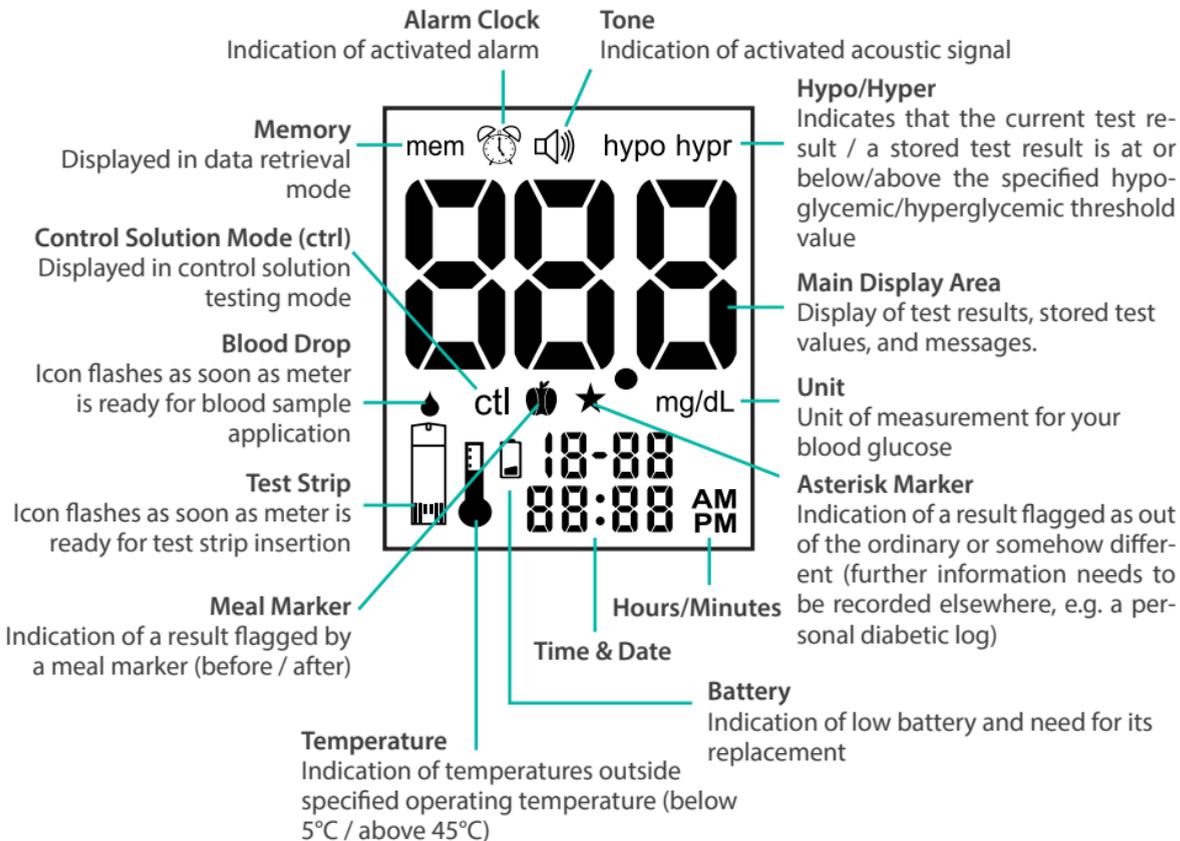The Wellion AMADEUS Blood Glucose Test Strip

Contact Points

Insert this end into the test strip port of the meter.

Sampling Area

Blood sample / Control solution application.

The Wellion AMADEUS Test Strip is a glucose specific, biosensor-based test strip that can test glucose in applicable blood samples in as little as 8 seconds requiring only a very small sample volume ($\geq 0,8\mu\text{L}$). The integrated underfill detection precludes performing a test with too little blood thus ensuring accurate and reliable results.

Test results are plasma calibrated allowing for direct comparison with lab results.

IMPORTANT

- The Wellion AMADEUS Blood Glucose Meter must only be used with Wellion AMADEUS Blood Glucose test strips. The use of other test strips may lead to incorrect test results or even cause damage to the device.
- The Wellion AMADEUS Blood Glucose Test Strips are sensitive to moisture and light , it is important to tightly close the vial cap of the test strip vial immediately after each use. DO NOT leave any test strips outside the vial while not in use.
- Carefully discard used test strips and lancets in accordance with local guidelines.
- DO NOT reuse test strips, as they are intended for single use only .

SETTING UP YOUR NEW SYSTEM

Inserting (or Changing) the Batteries

The batteries need to be inserted before using your Wellion AMADEUS Blood Glucose Meter for the first time or whenever the battery icon  appears on the meter display screen.


Material you will need:

2 x CR 2032 3V Lithium coin cell batteries

Your Wellion AMADEUS Blood Glucose Meter

Step 1: Switch off the meter. Remove the battery cover on the back of the meter by pushing the tab and pulling the door up. Remove the old batteries.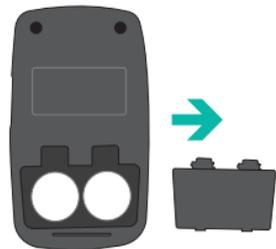

Step 2: Insert the new batteries with the "+" side up. They do not snap into place but rest on the metal contact. The door holds the batteries down. Put the battery door back into place and snap it closed.

NOTE

- After changing the batteries, the meter automatically prompts you to check time and date when it is turned on again either by inserting a test strip or pressing . If the settings are correct, press  for confirmation. If time and date are not correct, proceed as described in the chapter “Setting the Clock”.
- Neither the date nor any stored data are deleted when the battery is changed.
- Discard used batteries according to your local guidelines.
- The meter uses 2 x 3V lithium batteries, coin cell CR2032. This type of battery can be found in many stores. Always keep spare batteries on hand.
- Be sure the batteries go in with the “+” side up.
- Remove the batteries if you are not going to use the meter for an extended period of time.

Setting the Clock


Material you will need:

Your Wellion AMADEUS Blood Glucose Meter

Start Set Up

Step 1: Press and hold  to turn on the meter.

Step 2: The flashing test strip icon appears on the meter display screen.

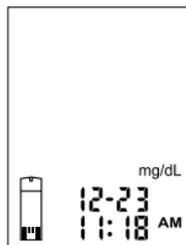
Step 3: Press **▲** twice until meter display screen shows “Set” and the clock icon  flashes.

Step 4 - **Set Year:**

Press **⏻** and the current year flashes. Use **▲** or **▼** to select the correct year. Press **⏻** to confirm your choice and move on to set the month.

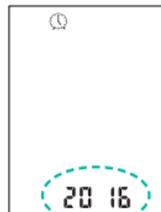Step 5 - **Set Month:**

The current month flashes. Use **▲** or **▼** to select the correct month. Press **⏻** to confirm your choice and move on to set the day.

Step 6 - **Set Day:**

The current day flashes. Use **▲** or **▼** to select the correct day. Press **⏻** to confirm your choice and move on to set the 12-hour or 24-hour time format.

Step 7 - Set 12-hour or 24-hour Time Format:

The time format flashes. Use \blacktriangle or \blacktriangledown button to select the time format of your choice.

Press ⏻ to confirm your choice and to move on to set the hour.

Step 8 - Set Hour

The current hour flashes. Use \blacktriangle or \blacktriangledown to select the correct hour.


Press ⏻ to confirm your choice and move on to set the minutes

Step 9 - Set Minutes

The current minutes flash. Use \blacktriangle or \blacktriangledown to select the correct minutes.

Press ⏻ to confirm your choice and exit time setup mode.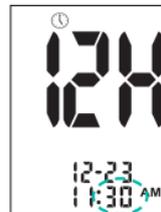

NOTE:

- Anytime during setting, you may either press ⏻ to exit, or insert a Wellion AMADEUS Blood Glucose Test Strip to start testing. The changes made so far are saved by the meter.


After setting up your clock, you can turn off the meter by holding and pressing ⏻ , continue to set your meter, or insert a test strip to start testing (please refer to the chapter "Testing Your Blood Glucose" or "Control Solution Testing").

Setting the Alarm

You can set up to three alarms on the Wellion AMADEUS Blood Glucose Meter. You must set the clock before being able to set any alarms. If an alarm goes off, a tune will be played for 30 seconds. Pressing  or inserting a test strip silences the alarm.

Step 1: Press and hold  to turn on the meter.

Step 2: The flashing test strip icon appears on the meter display screen.

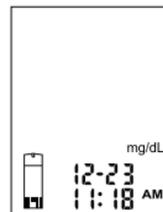Step 3: Press  or  until meter display shows "Set" and  starts to flash. Press  to enter the alarm setting mode.

Step 4: The display screen indicates alarm 1 status (per default alarm status is OFF). You can turn it on by pressing  or . Press  to confirm your choice and move on to set hours.


Step 5: The hours flash. Press \blacktriangle or \blacktriangledown to select the hour. Press ⏻ to confirm your choice and move on to set minutes.

Step 6: The minutes flash. Press \blacktriangle or \blacktriangledown to select the correct minutes. Press ⏻ to confirm your choice and move on to set alarm 2.

Step 7: Set alarm 2 and 3 by following steps 4 through 6. After you have finished setting up alarm 3, press ⏻ to exit the alarm setup mode.

Start testing by inserting a Wellion AMADEUS Blood Glucose Test Strip (please refer to the chapter “Testing Your Blood Glucose” or “Control Solution Testing”), or press and hold ⏻ to turn off the meter.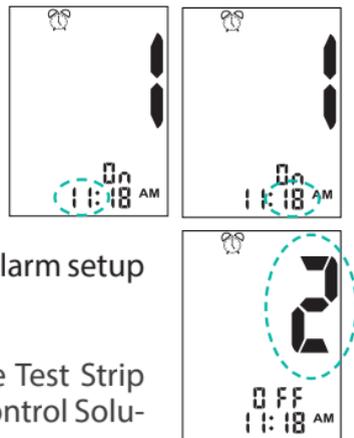


NOTE:

- Alarm will follow the time format (12-hour or 24-hour) predefined in your personal set up.
- Alarm settings will not be deleted when batteries are changed.
- Silencing the alarm does NOT deactivate it. If you do not actively deactivate it, it will go off at the same time on the next day.

Turning ON/OFF Tone Sound

The beep of the Wellion AMADEUS Blood Glucose Meter is activated by default. There is one beep whenever you switch the meter on or off and when a countdown comes to an end or to indicate that you have applied enough blood to perform a test.

The beep sounds twice in case of “hypo”, “hypr”, “HI” or “LO” warnings.

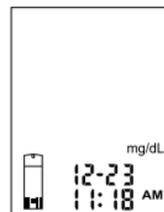Step 1: Press and hold  to turn on the meter.

Step 2: The flashing test strip icon appears on the meter display screen.

Step 3: Press  or  until meter display screen shows “Set” and the speakers icon  starts to flash .

Step 4: Press  to enter tone setting. Use  or  to turn on/off the tone.

Step 5: Press  to exit once you have made your selection.

NOTE

- Anytime during setup, you may either press  to exit or insert a test strip to start testing. The changes you made so far are saved by the meter.

Start testing by inserting a Wellion AMADEUS Blood Glucose Test Strip (please refer to the chapter “Testing Your Blood Glucose” or “Control Solution Testing”), or press and hold  to switch off the meter.

Setting Warning Values

Your Wellion AMADEUS Blood Glucose Meter allows you to set your high and low blood glucose threshold (hyperglycemia and hypoglycemia values).

Based on criteria established, the LCD screen will show “hypo” if your blood glucose test result is below your low glucose threshold value (hypoglycemia level) or the LCD screen will show “hyper” if your blood glucose test result is above your high glucose threshold level value (hyperglycemia level). Please consult your physician or healthcare provider on your individual recommendation for the Hypo and Hyper alarm thresholds.

HYPO
(hypoglycemia)
warning

HYPER
(hyperglycemia)
warning

IMPORTANT

- Do not change or discontinue your medication based on this feature, always consult your physician or healthcare provider before changing or discontinuing medication.
- The Hypo/Hyper warning of the Wellion AMADEUS Blood Glucose Meter is activated by default. The preset threshold value is 10 mmol/L (180 mg/dL) for Hyper and 3,9mmol/L (70 mg/dL) for Hypo. Follow the steps below to set your individual Hyper and Hypo threshold values.

Material you will need:

Your Wellion AMADEUS Blood Glucose Meter


Step 1: Press and hold  to turn on the meter.

Step 2: The flashing test strip icon appears on the meter display screen.


Step 3 - **Setting up hypo (hypoglycemia) warning value**

Press  or  until the meter display screen shows "Set" and "hypo" flashes.

Step 4: Press  to enter Hypo setup.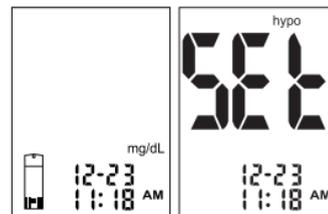

Step 5: Press \blacktriangle or \blacktriangledown to select the desired value. Pressing and holding either \blacktriangle or \blacktriangledown enables the fast forward function allowing you to move through the numbers at an accelerated pace. Press ⏻ to confirm and exit Hypo setup mode thus returning to the meter home screen.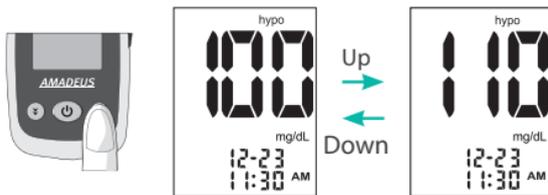


Step 6 - Setting up hyper (hyperglycemia) warning value

Press \blacktriangle or \blacktriangledown until the main display screen shows "Set" and "hypr" flashes.

Step 7: Press ⏻ to enter Hyper setup.

Step 8: Press \blacktriangle or \blacktriangledown to select the desired value. Pressing and holding either \blacktriangle or \blacktriangledown enables the fast forward function allowing you to move through the numbers at an accelerated pace. Press ⏻ to confirm and exit Hyper setup mode thus returning to the meter home screen.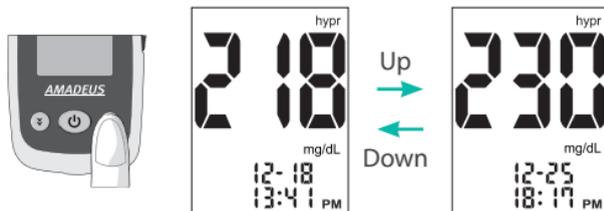

After setting up the Hypo and Hyper threshold values, you can either press and hold ⏻ to turn off the meter, or insert a test strip to start testing (please refer to the chapter "Testing Your Blood Glucose" or "Control Solution Testing").

Enable/Disable QC (Quality Control) Lock Mode

The Wellion AMADEUS Blood Glucose Meter offers the option of a QC(= Quality Control) lock mode. When the QC lock mode is on and no control test has been performed in the past 24 hours, the message “qC” flashes on the screen indicating the requirement to perform a control test before being able to run another glucose test.

Material you will need:

Your Wellion AMADEUS Blood Glucose Meter

Step 1: Press and hold  to turn on the meter.

Step 2: The flashing test strip icon appears on the meter display screen.

Step 3: Press  or  until the meter display screen shows “Set” and “ctl” flashes.


Step 4: Press  to enter QC lock setting. Use  or  to turn on/off QC lock.

Step 5: Press  to confirm and exit QC lock setup mode thus returning to the meter home screen.

Start testing by inserting a Wellion AMADEUS Blood Glucose Test Strip (please refer to the chapter “Testing Your Blood Glucose” or “Control Solution Testing”), or press and hold  to turn off the meter.

If the QC lock is activated and no control test has been done within the last 24 hours, the screen shows the flashing message “qC” when you insert a test strip. You are not able to run glucose tests without running a control test first.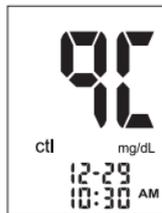

Press  to select control level (L1, L2 or L3), please refer to the chapter “Control Solution Testing” for further information.

CONTROL SOLUTION TESTING

Performing a Glucose Control Solution Test

The purpose of the control solution testing is to validate the performance of the Wellion AMADEUS Blood Glucose Meter using the control solution with a known range of glucose. Control solutions are NOT included in the Wellion AMADEUS Blood Glucose Meter kit, please contact the authorized representative for purchasing information.

Control solution testing is recommended whenever

- Using the meter for the first time
- You open a new vial of Wellion AMADEUS Blood Glucose Test Strips
- You left the cap of the test strip vial open over a longer period of time
- You dropped the meter
- You suspect that your Wellion AMADEUS Blood Glucose Meter and Wellion AMADEUS Blood Glucose Test Strips are not working properly
- The blood glucose test results do not reflect how you feel
- You want to practice the testing procedure

IMPORTANT

- Only use the Wellion AMADEUS Glucose Control Solution (Level 1, Level 2 and Level 3) together with the Wellion AMADEUS Blood Glucose Test Strips. Other brands of control solution will produce inaccurate results.
- Always check the expiration date . DO NOT use control solutions if expired.
- Record the opening date on the bottle whenever you open a control solution for the first time. After opening for the first time a control solution can be used for 3 months. Dispose of the bottle at the end of this period (according to your local regulation).
- DO NOT FREEZE or REFRIGERATE. Store the control solutions at 4°C-30°C.


Material you will need:

Wellion AMADEUS Control Solution (Level 1, Level 2 or Level 3)

Your Wellion AMADEUS Blood Glucose Meter

A new Wellion AMADEUS Blood Glucose Test Strip

Step 1: Insert a Wellion AMADEUS Blood Glucose Test Strip into the meter with the blood sample reaction zone facing up. Make sure the test strip contact points are inserted all the way into the meter.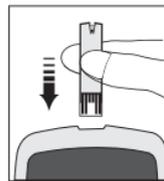

IMPORTANT

- Do not use a test strip that has expired. Always check the expiration date  which is printed on the test strip vial before starting a test.
- Use each test strip immediately after removing it from the vial.
- After removing a test strip from the vial, close the vial immediately and make sure it is tightly sealed.
- Keep away from direct sunlight and heat . Store the test strip vial in a dry, cool place.
- Record the “date opened” on the test strip vial label when you open it for the first time. The vial and any remaining test strips need to be discarded after 6 months starting from the date of opening.
- Make sure that the ambient testing temperature is within a range of 5°C-45°C. If you expose the device to temperature fluctuations please allow 10 to 15 minutes for it to adapt to the changed conditions. Your meter will not start testing if it detects an out-of-range temperature.



- Foil-wrapped test strips must NOT be open using scissors, as these might damage the test strip.
- This is an auto-coding meter. You do not need to insert a code card.
- You see a flashing blood drop icon if the test strip was inserted correctly into the test strip port.

Step 2: Wait until the flashing blood drop and test strip icons appear on the lower left of the meter display screen.

Step 3: Press \blacktriangledown to enter the control mode. You see “ctl” on the lower left of the screen. Use \blacktriangle or \blacktriangledown to select control level (L1, L2 or L3).

Step 4: Squeeze a drop of glucose control solution (Level 1, Level 2 or Level 3) onto a clean, dry, non-absorbent surface. Do not apply control solution to the test strip directly from the bottle. Close the control solution bottle immediately after use.

Step 5: Hold the meter and dip the end of the test strip into the drop of control solution. The control solution will be automatically pulled into the reaction area of the test strip.

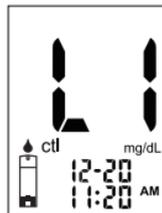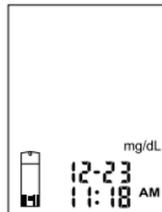Step 6: After successful sample application a countdown of 8 seconds starts, followed by the display of the control solution testing result. This result is clearly identified as a control result by the display of “ctl”.

Step 7: Compare the reading on the screen to the control range printed on the test strip vial. If the reading does not fall within the control range printed on the test strip vial, please refer to the chapter “Control Solution Trouble Shooting”.

NOTE

- Control solution testing results are saved in the meter's memory flagged as “ctl” results.
- Different vials or boxes of test strips may have different control ranges.
- Close the bottle containing the control solution immediately after use.

Step 8: Remove the used test strip either by pushing the ejection switch or by pulling it out manually and throw it into a proper waste basket. Insert a new test strip to perform the next test, or press and hold  to turn off the meter.

Perform testing of other control solution levels by once again going through the above described steps.

IMPORTANT

- Do not reuse test strips. 

Control Solution Trouble Shooting

If your control solution result is out of range (too high or too low), it may be due to one of the following reasons:

Possible Causes:	What you can do...
<ul style="list-style-type: none">• Use of an incompatible control solution• Use of an expired or contaminated control solution or of expired or damaged test strips• Meter malfunction• Improper storage of the control solution	<ul style="list-style-type: none">• Make sure you are using Wellion AMADEUS Glucose Control Solution (Level 1, Level 2 and Level 3).• Make sure the ambient testing temperature is within the range of 5°C-45°C.• Check the expiration and opening date on both the control solution bottles as well as on the test strip vials.• Repeat the test using a new test strip. If the result is still out of range, use a new bottle of control solution and retest.• Please contact your local distributor if you require further help.

TESTING YOUR BLOOD GLUCOSE

Materials you will need:

Your Wellion AMADEUS Blood Glucose Meter

A new Wellion AMADEUS Blood Glucose Test Strip

Lancing device with a sterile, unused lancet (not included in the set) or a safety lancet

Before you start, make sure that:

- your meter is set up properly. Please refer to the chapter “Setting Up Your New System” for more details.
- your hands have been thoroughly washed with soap and warm water, and that they are completely dry again.
- You are testing at an ambient temperature within the range of 5°C-45°C. Your meter does not test outside of this range but displays the error message . Move the meter to an area meeting required testing conditions and wait for 10 to 15 minutes before testing again.
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected appropriately.

Preparing your Lancing device

CAUTION


- Lancets are for single use only. ⓧ Always use a new, sterile lancet each time you perform the test.
- DO NOT share your lancing device or lancets with other people. Sharing or reusing lancets can lead to disease transmission.
- Dispose them according to local government regulations to avoid injury or contamination.
- If alcoholic wipes are used to cleanse the fingers, make sure that the area is dry before the blood sample is taken.

Step 1: Wash your hands with soap and warm water and dry them thoroughly. Warm water stimulates blood flow to the fingers making it easier to obtain a sample.

Step 2: Let your arm hang down for 10 to 15 seconds massaging through the wrist, palm and then finger. This can also stimulate the blood flow.

Step 3: Unscrew the adjustable cap of the lancing device and insert the lancet by pushing down firmly until it is fully seated.

Step 4: Twist the protective cover off from the lancet. Replace the adjustable cover back onto the lancing device.
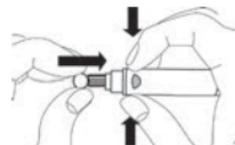


Step 5: Adjust the puncture setting on the adjustable cap for the puncture depth level (1 is the lightest and 10 is the deepest).

Step 6: With one hand holding the lancing device, pull back the cocking handle with your other hand until you hear a click sound and the lancing device is ready.

The release button should now be coloured red to indicate the lancing device is ready for blood sampling.

If the release button is not red, pull the cocking handle again until the release button changes the colour to red.

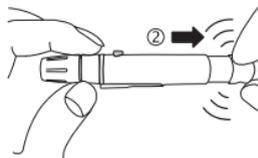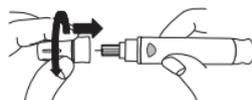TIP

- To avoid soreness, select a site on the side of your fingertip. To avoid calluses, choose a different site each time when taking a blood sample.

Important Information on Alternative Site Testing (AST)

The Wellion AMADEUS Blood Glucose Meter can test blood glucose from sites other than your fingertip such as palm or forearm (alternative site testing or AST). Alternative site testing can be less painful than fingertip testing, but because of the physiological difference between your fingertip and palm or forearm¹, AST results may be significantly different than results from fingertip testing. Always consult with your doctor or healthcare professional before using AST.

AST should only be used:

- In a pre-meal or fasting state (more than 2 hours since last meal)
- 2 hours or more after insulin application
- 2 hours or more after exercise

Alternative site measurements should never be used to calibrate continuous glucose monitors (CGMs).

Alternative site measurements should never be used for insulin dosing calculations.

AST SHOULD NOT be used when:

- You have hypoglycemic unawareness (you are not able to tell if you have low blood sugar).
- Within 2 hours of a meal, exercise or medication.
- You operate machinery or drive a car.
- You are sick.


- You think your blood glucose is low.
- Your AST results do not reflect the way you feel.
- You suspect hyperglycemia.
- Your blood glucose levels fluctuate frequently.

Consult with your healthcare professional to decide if AST is right for you.

1. Shu M, Osamu F, Kazuhiro H, Yoshihito A: Hypoglycemia Detection Rate Differs Among Blood Glucose Monitoring Sites. *Diabetes Care* 28(3):708–709, 2005

Performing a Blood Glucose Test

Step 1: Insert a test strip to switch on the meter.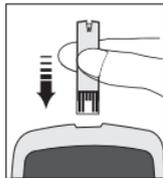

IMPORTANT

- Do not use a test strip that has expired. Always check the expiration date  which is printed on the test strip vial.
- Use each test strip immediately after removing it from the vial.
- Do not use wet or damaged test strips.
- Test strips need to be kept away from direct sunlight and heat . Store test strips in a dry, cool place.
- Record the “date opened” on the vial label. The vial and any remaining test strip need to be discarded after 6 months from the date of opening.
- An insufficient amount of blood may cause incorrect results.



- Correct test strip insertion and meter's readiness for testing is indicated by the flashing blood drop icon.

Step 2: Wait until the flashing blood drop and test strip icons appear on the lower left of the meter display screen.

Step 3: How to take a blood sample:

For Fingertip Testing:

Hold the primed lancing device firmly against the side of your fingertip and press the release button to create a puncture.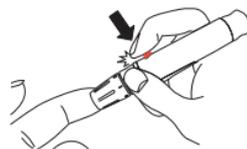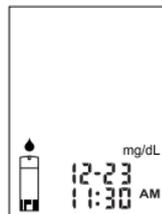

TIP

- Gently massage your hand and finger toward the puncture site to form a drop of blood. Do not "milk" or squeeze around the puncture site.
- Puncturing the side of your fingertip helps to avoid soreness. To avoid calluses, choose a different lancing site each time.

For Testing on Palm or Forearm:

Always consider mandatory prerequisites for AST (please refer to the chapter “Important Information on Alternative Site Testing (AST)”). Select a puncture site on forearm or palm. Avoid veins, hair, moles, bone and tendons.

For palm testing, it is not necessary to rub the skin or pump the device. Press the lancing device firmly against the puncture site and then press the release button.


For forearm testing only, rub testing area vigorously until it feels warm to increase blood flow. Press the lancing device firmly against the puncture site and then press the release button. Keep the device in constant contact with the skin and apply and release pressure up and down 2–3 times without lifting the device.

When the blood is about the size of a pen tip lift the lancing device straight up without smearing the blood.

IMPORTANT

- If results from AST do not reflect the way you feel, use fingertip testing instead.


Step 4: Gently dip the test strip into the drop of blood at a slight angle. The test strip acts like a straw to pull the blood in. Keep the test strip in contact with the blood drop until the meter beeps to indicate there is enough blood to test.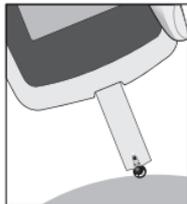

Step 5: After successful sample application a countdown of 8 seconds starts, followed by the display of your blood glucose testing result.

Step 6 - Select the before/after meal marker:


Before you remove the test strip and while the test result is still displayed, there is the option to flag it with one of the following informations: "before meal" (🍏) / "after meal" (🍎) / no information. In order to do so, press either ⬆ or ⬇ to select the appropriate marker (blank if the test result does not apply). Press ⏻ to confirm your choice and move on to select asterisk marker.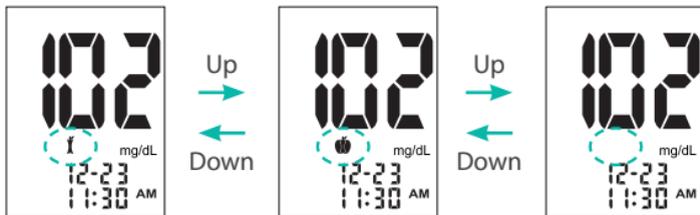

Step 7 - Select the asterisk marker:

After assignment of meal markers, a flashing ★ icon appears in the middle of the screen.

You can then flag your result to remind you of an event out of the ordinary such as an AST result or exercise. Use ⬆ or ⬇ to turn the option on/off, and press ⏻ to confirm your choice. The result will be saved together with its assigned markers.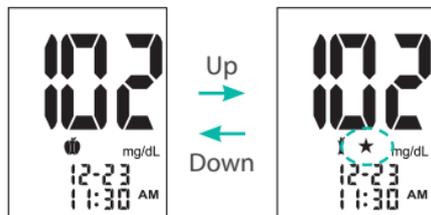

Test results will be stored without any markers if the strip is removed prior to selection.

CAUTION

- If “HI” or “LO” is displayed, your blood glucose level may be beyond the meter measurement range (above 600 mg/dL (33,3 mmol/l) or below 20 mg/dL (1,1 mmol/l)). Test again using fingertip testing, DO NOT test on palm or forearm. If you still get the same result, call your physician or healthcare professional immediately.

NOTE

- “hypo” or “hypr” icons may appear on the screen depending on your preset threshold limits. Please refer to the chapter “Setting Warning Values”.

Step 8: Remove the used test strip either by pushing the ejection switch or by pulling it out manually and throw it into a proper waste basket. Insert a new test strip to perform the next test, or press and hold  to switch off the meter.

Step 9: Remove the used lancet from your lancing device. You can use the lancet ejector to prevent needlestick injuries. Push the lancet ejector forward with the thumb and discard the lancet in accordance with your local guidelines.

IMPORTANT

- Used lancets and strips are biohazardous materials and can transmit blood-borne diseases. Please follow your local healthcare provider’s recommendation regarding proper disposal of used lancets and strips.

UNDERSTANDING YOUR BLOOD GLUCOSE TEST RESULTS

Blood glucose values vary depending on food intake, medication, health, stress, and exercise. The ideal ranges for adults without diabetes are²:

- less than 100 mmol/L (5,6 mg/dL) before meals
- less than 140 mmol/L (7,8 mg/dL) after meals

It is important to consult with your physician or healthcare professional to determine an appropriate target range for you.

What to do if you get a high or low blood glucose reading

If the Meter displays „HI“ or „LO“, or if you get a result that is outside specified hyper- and hypoglycemia values AND you feel ill:

- Treat your diabetes according to the instruction from your doctor and/or consult with your healthcare provider.

If the Meter displays „HI“ or „LO“, or if you get a result that is outside specified hyper- and hypoglycemia values AND you DO NOT feel ill:

- Test your meter with a control solution, please refer to chapter “Control Solution Testing”.
- Test again using fingertip.
- If you still get a high or low reading, contact your health care professional.

IMPORTANT

- Inaccurate results may be obtained in severely hypotensive individuals or patients in shock.

- Inaccurate low results may be obtained in individuals experiencing a hyperglycemic-hyperosmolar state, with or without concomitant ketosis.
- Critically ill patients should not be tested.

RETRIEVAL OF RESULTS FROM THE DATA MEMORY


Your Wellion AMADEUS Blood Glucose Meter can store up to 1000 test and control results together with date and time. You need to set date and time before using your meter's memory function - please refer to the chapter "Setting Up Your New System". The meter does not save any test or control results if the date and time are not set.

Material you will need:

Your Wellion AMADEUS Blood Glucose Meter


Step 1: Press and hold  to switch on the meter.

Step 2: The flashing test strip icon appears on the meter display screen.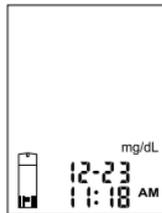


Step 3: Use  or  to enter the settings menu and until "mem" flashes.

Step 4: Press  to confirm your choice.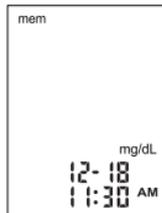

Step 5: The most recent test result will appear on screen. Press \blacktriangle to view your result from the most recent to the oldest.

Pressing and holding either \blacktriangle or \blacktriangledown enables the fast forward function allowing you to move through the stored results at an accelerated pace.

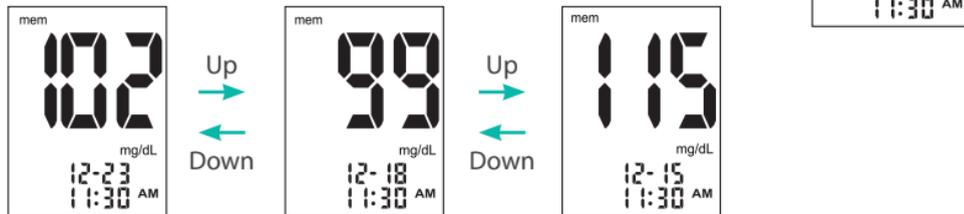After you finish viewing the memory, either start testing by inserting a test strip (please refer to the chapter “Testing Your Blood Glucose” or “Control Solution Testing”) or press and hold ⏻ to turn off the meter.

NOTE

- Control testing results are flagged by a “ctl” icon on the lower left.
- Hypoglycemic readings and hyperglycemic readings are displayed together with the corresponding icons.
- Results that were classified as “HI” or “LO”, will be stored in the memory as 600 mg/dL (33,3 mmol/l) or 20 mg/dL (1,1 mmol/l) without any markers.
- If there is an error message, please refer to the chapter “Troubleshooting”.

Upload Data

Insert Wellion AMADEUS USB cable into the data port, the PC mode shows up as the right image.

Follow the Data Management System (DMS) instruction manual to upload data. For purchase information of DMS, please contact your local distributor.

NOTE

- The upload feature should only be used with a DMS product that specifically identifies the Wellion AMADEUS Blood Glucose Meter as a meter that is compatible with their DMS.

MAINTENANCE

Maintaining your Wellion AMADEUS Blood Glucose Meter is easy. Follow these simple guidelines to keep your meter working properly and in good conditions.

Cleaning the Meter

- To clean the meter thoroughly wipe the entire surface with a damp lint-free cloth and mild soap or detergent to remove any possible dirt, dust, blood or other body fluids.
- Do not allow water to enter the Wellion AMADEUS Blood Glucose Meter. Never immerse the meter or hold it under running water.

- Do not use glass or household cleaners on the meter.
- Do not try to clean the test strip holder.
- Do not contaminate the strip holder with blood or control solution.

Storage and Precautions

- Always handle the meter with care; severe shock, such as dropping the meter, could damage the electronics.
- Both meter and test strips are designed to be used within temperature ranges of 5°C and 45°C and ranges of relative humidity of 20%-90%
- Avoid leaving the meter in extremely hot or cold places, e.g. in close proximity to a heat source or in an extremely hot or cold car.
- Do not store or use the meter or test strips where they may be exposed to high humidity levels, e.g. in a bathroom or in a kitchen.
- Always close the vial cap immediately after removing a test strip and make sure it is then closed tightly.
- Warranty does not apply to a meter that has been disassembled, altered, misused, tampered with or abused.
- Do not use this meter in a dry environment, especially if synthetic materials are present. Synthetic clothes, carpets, etc., may cause damaging static discharges in a dry environment.
- To avoid potential electromagnetic or other interference, keep meter away from electromagnetic radiation sources such as X-rays or MRI.
- Dispose of the meter according to your local regulations for correct disposal.

TROUBLESHOOTING

This section explains the most common display screen messages and error codes you may encounter when using your Wellion AMADEUS Blood Glucose Meter and Wellion AMADEUS Test Strips.

Message	Meaning	Resolution
	Damaged meter electronics or damaged test strip	<ul style="list-style-type: none"> Remove the batteries and switch the meter on again. Remove the test strip and replace it with a new one. If the problem persists, contact the local distributor.
	Used or contaminated test strip	<ul style="list-style-type: none"> Repeat the test with a new test strip. Wait until you see the flashing blood drop icon before you add blood or control solution sample.
	Not enough blood sample on the test strip to start	<ul style="list-style-type: none"> Remove the test strip and repeat the test with a new test strip. Please refer to the chapter "Testing Your Blood Glucose".

	<p>Premature sample application</p>	<ul style="list-style-type: none"> Remove the test strip and repeat the test with a new test strip. Apply blood only AFTER the appearance of the flashing blood drop icon. Please refer to the chapter "Testing Your Blood Glucose".
	<p>No memorized results in the meter</p>	<ul style="list-style-type: none"> Check if the date and time on your meter is set up. Please refer to the chapter "Setting Up Your New System". Start testing your blood glucose, please refer to the chapter "Testing Your Blood Glucose"
	<p>Test result is higher than 600 mg/dL (33,3 mmol/L)</p>	<ul style="list-style-type: none"> Wash and thoroughly dry your hands and repeat the test using a new test strip. If the result is still "HI", contact your physician or healthcare professional immediately.
	<p>Test result is lower than 20 mg/dL (1,1 mmol/L)</p>	<ul style="list-style-type: none"> Wash and thoroughly dry your hands and repeat the test using a new test strip. If the result is still "LO", contact your physician or healthcare professional immediately.

	Temperature out of range	<ul style="list-style-type: none">• Make sure that the ambient testing temperature is within the range of 5-45°C. If meter and / or test strips have been exposed to substantial change in temperature, please wait for 10 to 15 minutes before restarting the measurement.
	Low battery	<ul style="list-style-type: none">• Please refer to the chapter „Inserting (or Changing) the Batteries“ for battery replacement.

SYMBOLS

(EXP) Expiration date (use-by date)

(LOT) Batch Code

Temperature limitations

Consult Instructions for Use

In Vitro Diagnostic Medical Device

Caution! Consult accompanying documents

Catalogue number

Keep away from sunlight/direct light

Do not reuse

Keep dry

Manufacturer

Authorised representative in the European Community

This product complies with the EC directives and bears the CE mark.

0197 indicates the number of notified body involved in production quality module

Indicates the entity importing the medical device into the locale

Production date

SPECIFICATIONS

Test Strips:	Wellion AMADEUS Blood Glucose Test Strip
Test Range:	20 - 600 mg/dL (1,1-33,3 mmol/l)
Calibration:	Plasma
Blood Sample Size:	≥ 0,8 µL
Hematocrit Range:	10-70%
Event management:	Before/After meal marker and asterisk marker
Warnings:	User configurable hypoglycemia and hyperglycemia warnings, underfill warning
Display Type:	LCD screen with back lighting
Memory:	1000 test results together with date and time
Dimensions:	105 x 60 x 14,5 mm
Weight:	51g (without batteries)
Batteries:	2 x CR 2032 3V Lithium coin cell battery
Battery Life:	1000 tests in continuous use or one year
Automatic Power-off:	After 2 minutes of inactivity
Operating Temperature:	5°C to 45°C
Operating Humidity:	20-90%
Alarms:	3
Storage/Transport Condition:	Meter at 4-50°C, 10-85 % RH Test strip at 4-30°C, 10-85% RH

For additional information, refer to the Wellion AMADEUS Blood Glucose Test Strip.

Electromagnetic Compatibility:

This equipment complies with EMC requirement of EN 61326-1/ EN 61326-2-6.

EU directive/classification: Directive 98/79/EC Annex II, List B;

MANUFACTURER'S WARRANTY

The manufacturer of the Wellion AMADEUS Blood Glucose Meter assumes warranty for material and manufacturing defects for a period of two years from date of purchase. This guarantee is lost when the device is not used as intended, has been poorly maintained or opened.

Warranty is limited to the repair of defective parts or - in accordance with the manufacturer - in replacement of the device. The right to cancel the purchase applies only if the replacement goods are also faulty. Other claims cannot be accepted. Warranty by MED TRUST is limited to the repair or replacement of the device and in no case can MED TRUST be held responsible for any collateral or consequential damages or losses.

The warranty is invalidated by damage from improper use, non-authorized repair or modification to the device.

Consumables are explicitly excluded from this warranty.

The warranty period cannot be extended.

wellion® **AMADEUS**

THE MEDICAL SERVICES COMPANY

Service and distribution:

MED TRUST Handelsges.m.b.H.

Gewerbepark 10,

7221 Marz, AUSTRIA

www.medtrust.eu; www.wellion.eu

Tel.: + 43 2626 64 190

Fax: + 43 2626 64 190 - 77

E-Mail: office@medtrust.at

OBELIS S.A.

Bd. Général Wahis, 53, 1030 Brussels, Belgium

Tel: +32.2.732.59.54, Fax: +32.2.732.60.03

mail@obelis.net

APEX BIOTECHNOLOGY CORP, No. 7, Li-Hsin Rd. V,

Hsinchu Science Park, Hsinchu, Taiwan

TEL: 886-3-5641952

info@apexbio.com, www.apexbio.com

**SUITABLE FOR SELF-TESTING /
ZUR SELBSTTESTUNG GEEIGNET**

CE 0197

**In Vitro Diagnostic
Medical Device**

W18-32DE/EN_20250103

© MED TRUST, Wellion and AMADEUS are registered trademarks.